Ulthera® DeepSEE Transducer  
Instructions for Use

*Table of Contents*

|                                         |    |                                    |     |
|-----------------------------------------|----|------------------------------------|-----|
| EN – English.....                       | 3  | LV – Latvian – Latviski.....       | 71  |
| BG – Bulgarian – Български .....        | 7  | MK – Macedonian – Македонски ..... | 75  |
| BS – Bosnian – Bosanski.....            | 11 | NL – Dutch – Nederlands.....       | 79  |
| CS – Czech – Český Jazyk.....           | 15 | NO – Norwegian – Norsk .....       | 83  |
| DA – Danish – Dansk .....               | 19 | PL – Polish – Polski .....         | 87  |
| DE – German – Deutsch .....             | 23 | PT – Portuguese – Português.....   | 91  |
| EL – Greek – Ελληνικά.....              | 27 | RO – Romanian – Românește.....     | 95  |
| ES – Spanish – Español .....            | 31 | RU – Russian – Русский.....        | 99  |
| ET – Estonian – Eesti Keel .....        | 35 | SK – Slovak – Slovenčina.....      | 103 |
| FI – Finnish – Suomi .....              | 39 | SR – Serbian – Српски.....         | 107 |
| FR – French – Français .....            | 43 | SV – Swedish – Svenska .....       | 111 |
| HU – Hungarian – Magyar .....           | 47 | TH – Thai – ไทย.....               | 115 |
| ID – Indonesian – Bahasa Indonesia..... | 51 | TR – Turkish – Türkçe.....         | 121 |
| IT – Italian – Italiano.....            | 55 | UK – Ukrainian – Українська.....   | 125 |
| JA – Japanese – 日本語.....                | 59 | VI – Vietnamese – TIẾNG VIỆT.....  | 129 |
| KO – Korean – 한국어 .....                 | 63 | ZH – Chinese (Trad.) – 中文 .....    | 133 |
| LT – Lithuanian – Lietuviškai .....     | 67 |                                    |     |

EN – English

Ulthera® DeepSEE Transducer  
Instructions for Use

*The Ulthera® DeepSEE Transducer is only to be used with the Ulthera® System. Please refer to the Ulthera® System Instructions For Use for proper usage of this transducer.*

Ulthera, Inc.  
6501 Six Forks Road  
Raleigh, NC 27615  
USA  
Phone: +1-877-858-4372  
Ultherapy.com

### Description

The Ulthera® DeepSEE Transducer is a reusable, multi-patient ultrasound transducer that allows the operator to visualize the skin and sub-dermal regions of interest for a specific treatment. The transducer directs acoustic waves to the treatment area heating tissue as a result of frictional losses during energy absorption and produces a discrete zone of coagulation.

### Indications for Use

The Ulthera® DeepSEE Transducer is only to be used with the Ulthera® System. Please refer to the Ulthera® System Instructions for Use for proper usage of this transducer.

**Contraindications**

The Ulthera® DeepSEE Transducer is only to be used with the Ulthera® System. Please refer to the Ulthera® System Instructions for Use for proper usage of this transducer.

**Precautions and Warnings**

The Ulthera® DeepSEE Transducer is only to be used with the Ulthera® System. Please refer to the Ulthera® System Instructions for Use for proper usage of this transducer.

**Patient Safety**

Warning: Ulthera® should not be used on a patient's eyes or with a technique that would allow ultrasound energy to reach the eye, such as on or over the eyelids or within the orbital rim.

**Storage, Handling and Disposal**

Store in a cool, dry place at room temperature. Storage Range 15°C (59°F) to 30°C (86°F) 15-95% R.H. 500 to 1060hPa (0.5 to 1.05 ATM). Protect from freezing.

Depleted transducers should be disposed of in accordance with federal, state and local regulations.

**Procedure for Using the Ulthera® DeepSEE Transducer**

See the Ulthera® System Instructions for Use

**CAUTION**

CAUTION: U.S. Federal Law restricts this transducer for sale by or on the order of a physician.

European Conformity

Keep Dry

Refer to Instruction Manual/Booklet

Catalogue Number

Recycle Packaging

Caution

Manufacturer

Fragile, Handle with Care

Warning

Storage Temperature Limit

Relative Humidity Limitation

CSA Certification Mark applies to use of transducer only with CSA-certified legacy Ulthera model UC-1.

Keep Electrical Waste Separate from Municipal Waste

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